



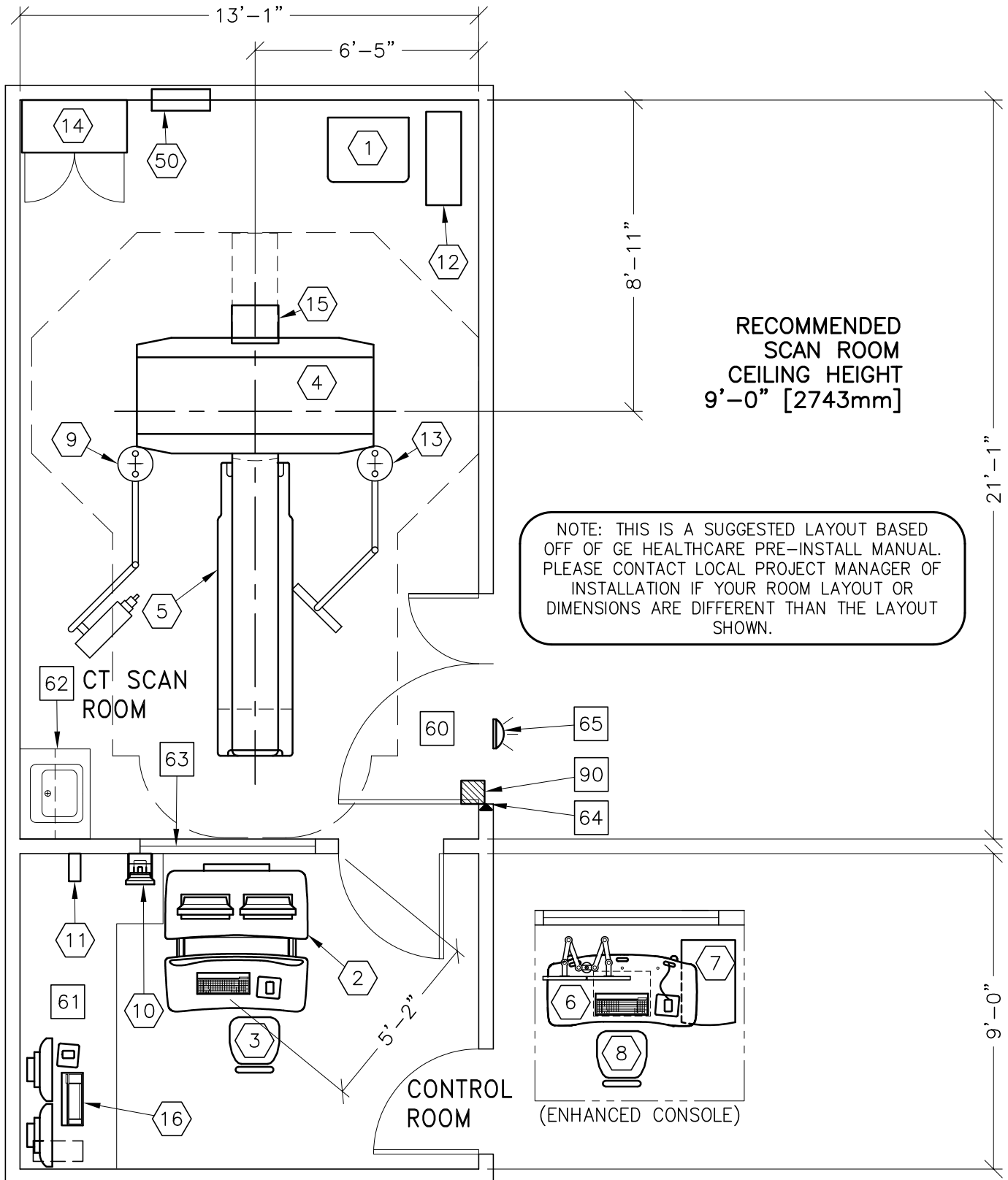

BRIGHTSPEED SELECT

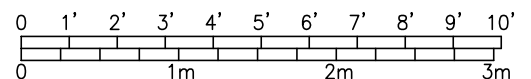

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
SUGGESTED:	20'-8" x 13'-0"	T.B.D.



SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 6-62

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	POWER DISTRIBUTION UNIT	771 lbs	5119 btu
②	1	OPERATOR'S CONSOLE / COMPUTER	469 lbs	8191 btu
③	1	OPERATOR'S CHAIR		
④	1	CT BRIGHTSPEED SELECT GANTRY	3899 lbs	18778 btu
⑤	1	PATIENT TABLE	705 lbs	682 btu
⑥	1	FREEDOM WORKSPACE SMALL TABLE	136 lbs	8191 btu
⑦	1	T. I. O. CABINET	194 lbs	
⑧	1	OPERATOR'S CHAIR		
⑨	1	INJECTOR HEAD ON OVERHEAD COUNTERPOISED SUSPENSION.	22 lbs	
⑩	1	INJECTOR CONTROL CONSOLE	8 lbs	
⑪	1	MAIN UNIT	13 lbs	
⑫	1	UPS SYSTEM	350 lbs	
⑬	1	LCD MONITOR	61 lbs	300 btu
⑭	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	90 lbs	
⑮	1	REAR CABLE COVER		
⑯	1	ADVANTAGE WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑰	1	MAIN DISCONNECT CONTROL CAT NO. E4502AB <IF A UPS IS NOT OR WILL NOT BE ORDERED, THE E4502AD CAN BE USED.>	90 lbs	

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)
 Allowable Temperature Change: 5° (F) per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 10,000 ft. above sea level

ROOM PLAN LOCATED ON PAGE 6-62

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
60		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
61		COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
62		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
63		LEAD GLASS WINDOW		
64		DOOR LIMIT SWITCH <REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
65		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABW-0F-XIU		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		