





BRIGHTSPEED ELITE, EDGE, EXCEL

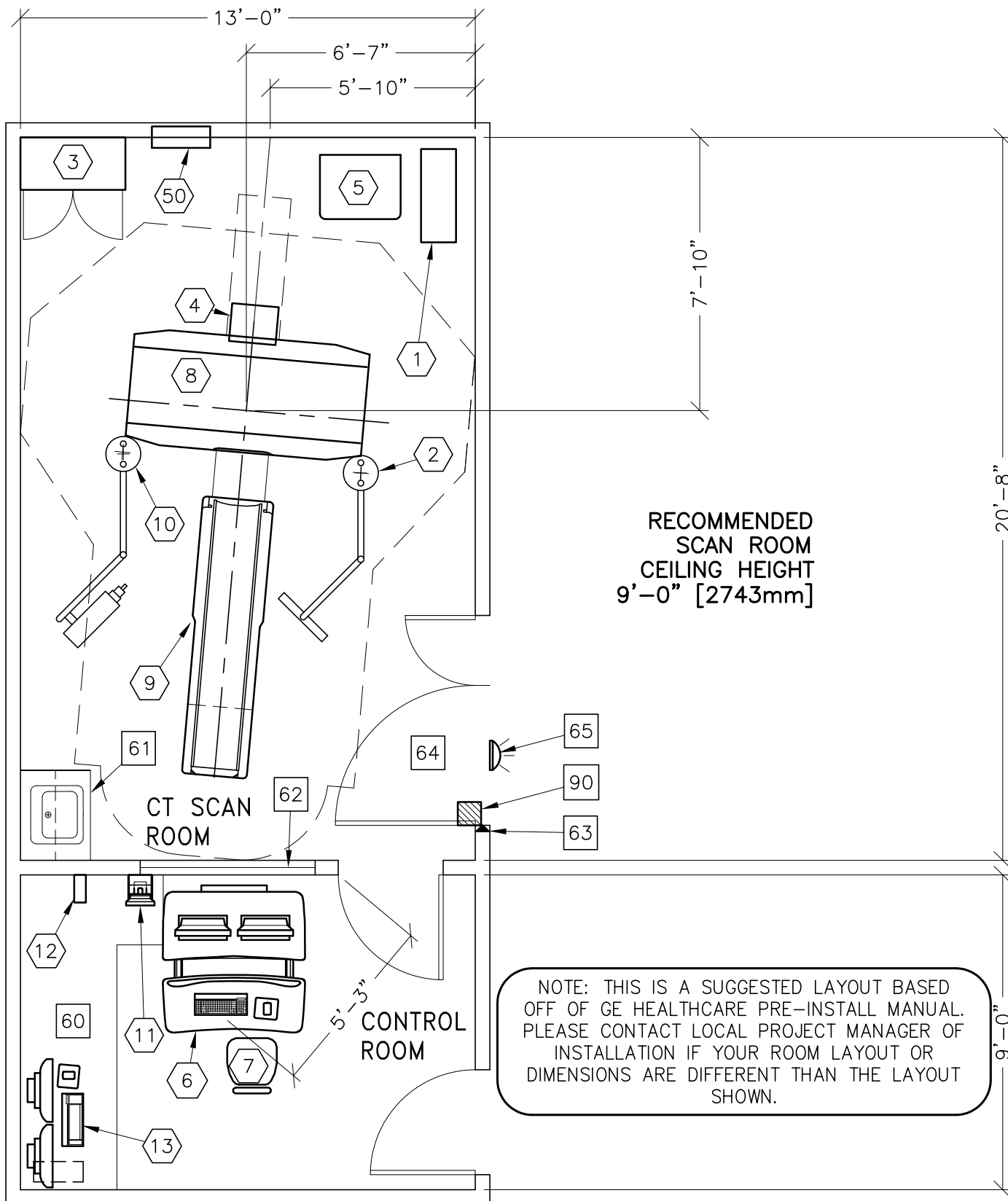

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL:	19'-3" x 12'-10" [5.8M x 3.9M]	8'-0" [2.4M]

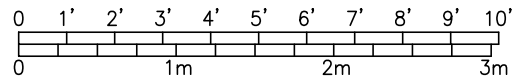

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 6-58

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	UPS SYSTEM	350 lbs	
②	1	LCD MONITOR	61 lbs	300 btu
③	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	90 lbs	
④	1	REAR CABLE COVER		
⑤	1	POWER DISTRIBUTION UNIT	771 lbs	5119 btu
⑥	1	OPERATOR'S CONSOLE / COMPUTER	535 lbs	8191 btu
⑦	1	OPERATOR'S CHAIR		
⑧	1	CT BRIGHTSPEED GANTRY	3899 lbs	18771 btu
⑨	1	PATIENT TABLE WITH EXTENDED TABLE TOP.	749 lbs	682 btu
⑩	1	INJECTOR HEAD ON OVERHEAD COUNTERPOISED SUSPENSION.	22 lbs	
⑪	1	INJECTOR CONTROL CONSOLE	8 lbs	
⑫	1	MAIN UNIT	13 lbs	
⑬	1	ADVANTAGE WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑤①	1	MAIN DISCONNECT CONTROL CAT NO. E4502AB <IF A UPS IS NOT OR WILL NOT BE ORDERED, THE E4502AD CAN BE USED.>	90 lbs	

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)
 Allowable Temperature Change: 15° (F) per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 7875 ft. above sea level

ROOM PLAN LOCATED ON PAGE 6-58

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
60		COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
61		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
62		LEAD GLASS WINDOW		
63		DOOR LIMIT SWITCH (REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)		
64		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
65		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		