





INNOVA BIPLANE IGS 620/630 ELECTROPHYSIOLOGY (EP)

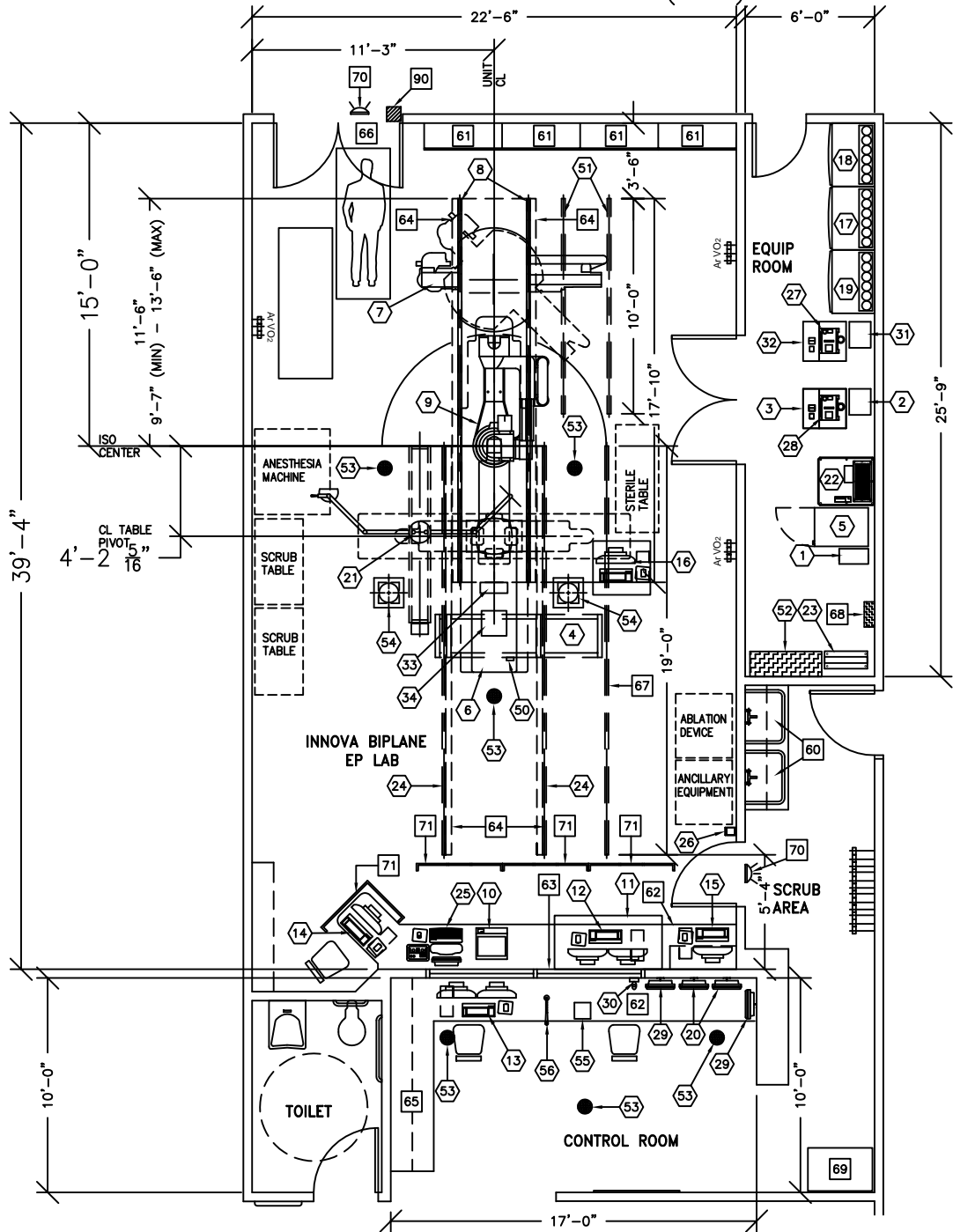

NOTE: THIS IS A SUGGESTED LAYOUT BASED OFF OF GE HEALTHCARE PRE-INSTALL MANUAL. PLEASE CONTACT LOCAL PROJECT MANAGER OF INSTALLATION IF YOUR ROOM LAYOUT OR DIMENSIONS ARE DIFFERENT THAN THE LAYOUT SHOWN.

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
RECOMMENDED:	38'-0" x 26'-2" [11.5m x 8.0m]	9'-4" [2.85m]

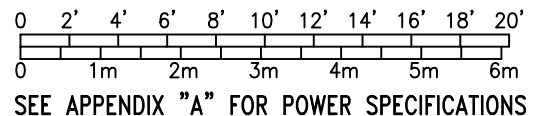

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 5-113

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	3 KVA UPS CABINET <LARGE DISPLAY SUBSYSTEM OPTION>	66 lbs	546 btu
②	1	AP COOLIX 4100 AUTOTRANSFORMER	66 lbs	238 btu
③	1	AP COOLIX 4100 WATER CHILLER	264 lbs	18730 btu
④	1	LARGE DISPLAY MONITOR ON SINGLE MONITOR SUSPENSION 7 ft. 9 in. INBOARD BRIDGE <MOUNT TWO GE MONITORS ON BACKSIDE OF LD MONITOR>	784 lbs	1706 btu
⑤	1	LARGE DISPLAY MONITOR CABINET	253 lbs	3412 btu
⑥	1	OMEGA V TABLE	1300 lbs	614 btu
⑦	1	LATERAL POSITIONER BRIDGE MOUNT ASSEMBLY MOUNTED FROM CEILING SUPPORTS	1421 lbs	4126 btu
⑧	2	LONGITUDINAL STATIONARY RAIL FOR LATERAL GANTRY INNOVA POSITIONER	68 lbs	
⑨	1	INNOVA POSITIONER <REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION>	1653 lbs	2416 btu
⑩	1	COLOR PRINTER		
⑪	1	WORKSTATION CART		
⑫	1	COMBOLAB CONSOLE INCLUDES MONITORS AND KEYBOARD	566 lbs	2935 btu
⑬	1	REMOTE MONITORING WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑭	1	CLINICAL WORKSTATION	46 lbs	682 btu
⑮	1	MAPPING WORKSTATION	46 lbs	682 btu
⑯	1	NURSING NOTES WORKSTATION	46 lbs	682 btu
⑰	1	LC/LP CABINET <C2>	621 lbs	4570 btu
⑱	1	LATERAL CABINET <C3>	703 lbs	2945 btu
⑲	1	AP FRONTAL CABINET <C1>	888 lbs	4413 btu
⑳	2	18 in. MONITOR ON WALL SUPPORT <AP LIVE & REFERENCE>	26 lbs	204 btu
㉑	1	COUNTERBALANCED EYE AND THYROID SHIELD WITH R96 LAMP	143 lbs	
㉒	1	UPS CABINET	1170 lbs	4061 btu
㉓	1	3 KVA UPS CABINET	81 lbs	546 btu
㉔	2	LONGITUDINAL STATIONARY RAIL FOR LCD MONITOR SUSPENSION	68 lbs	
㉕	1	OPERATORS CONSOLE	22 lbs	546 btu
㉖	1	XR BUZZER <LOCATED ABOVE CEILING>	2 lbs	

ENVIRONMENT

Ambient Operation Temperature: 68°-77° F, (20°-25° C)
 Allowable Temperature Change: Not Available
 Humidity: 30%-75% non condensing
 Allowable Humidity Change: Not Available
 Max - Altitude: 9842 ft. (3000 M) above sea level

ROOM PLAN LOCATED ON PAGE 5-113

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
27	1	LATERAL DETECTOR CHILLER	33 lbs	709 btu
28	1	AP DETECTOR CHILLER	33 lbs	709 btu
29	2	18 in. MONITOR ON WALL SUPPORT <LATERAL LIVE & REFERENCE>	26 lbs	204 btu
30	1	BOLUS CHASE HANDSWITCH	2 lbs	
31	1	LATERAL COOLIX 4100 AUTOTRANSFORMER	66 lbs	238 btu
32	1	LATERAL COOLIX 4100 WATER CHILLER	264 lbs	18730 btu
33	1	TRAM NET RACK	8 lbs	
34	1	CLAB 2 PLUS AMPLIFIER	24 lbs	204 btu
50	1	VITALINQ MICROPHONE <ONE ON MONITOR BRIDGE IN EXAM ROOM>		
51	2	CABLE DRAPE RAIL FOR LP POSITIONER		
52	1	INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.	899 lbs	2215 btu
53	6	VITALINQ SPEAKER		
54	2	SKYTRON LIGHTING UNIT	50 lbs	341 btu
55	1	VITALINQ CONSOLE		
56	1	VITALINQ MICROPHONE		
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		SCRUB SINK		
61		CATHETER CABINETS		
62		COUNTER TOP FOR EQUIPMENT- SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
63		CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING WINDOW.		
64		BEARING BLOCK OUTLINE, SEE S1 FOR MORE INFORMATION.		
65		COUNTER TOP W/BASE & WALL CABINETS		
66		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
67		CABLE DRAPE RAIL.		
68		150-AMP LOCAL SERVICE DISCONNECT FOR LOCK-OUT/ TAG-OUT CAPABILITY. <MAY BE A FUSED DISCONNECT, CIRCUIT BREAKER OR SAFETY SWITCH.>		
69		CUSTOMER SUPPLIED STORAGE CABINET		
70		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU		
71		MOBILE RADIATION SHIELD		
90		REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION - RW120-10-RL FOR WARNING LIGHT CONTROL ONLY <AVAILABLE FROM GEXPRO> 1-800-200-9760 <317-554-3805>, EXTENSION 3825.		