



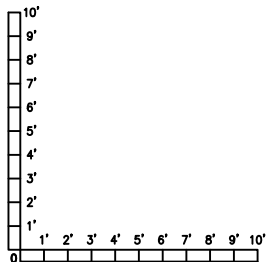

HYBRID OR

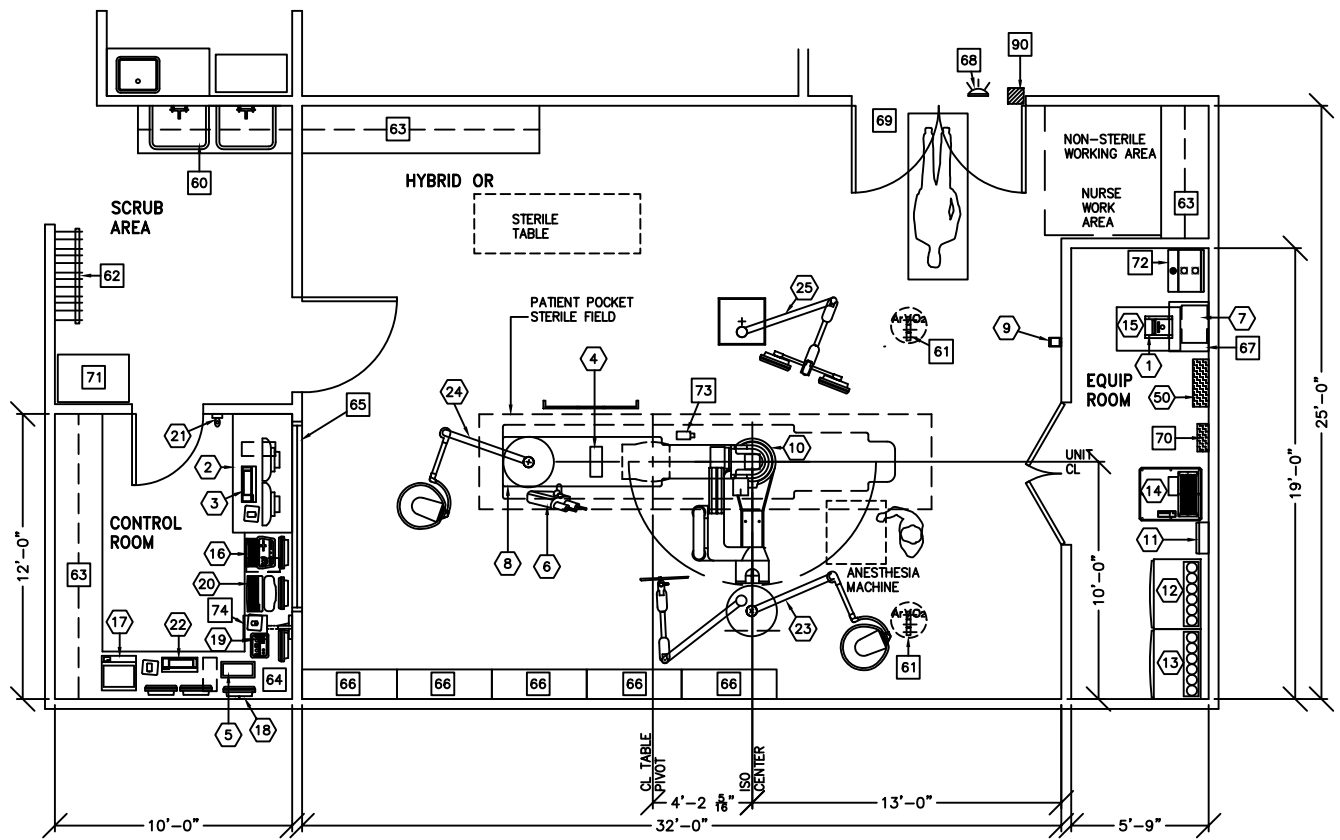

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
ABSOLUTE MINIMUM:	22'-6" x 13'-9" [6.89m x 4.23m]	9'-6" ±0.2 [2.84m]

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 5-93

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	DETECTOR CHILLER	33 lbs	706 btu
②	1	WORKSTATION CART		
③	1	MAC LAB CONSOLE, INCLUDES MONITORS AND KEYBOARD	181 lbs	2935 btu
④	1	TRAM NET RACK	8 lbs	
⑤	1	REMOTE CONTROL FOR INJECTOR	4 lbs	
⑥	1	INJECTOR HEAD ON TABLE RAIL	15 lbs	
⑦	1	INJECTOR ELECTRONICS	37 lbs	320 btu
⑧	1	OMEGA IV/V TABLE WITH ROTATING TOP	1750 lbs	614 btu
⑨	1	XR BUZZER <LOCATED ABOVE CEILING>	2 lbs	
⑩	1	INNOVA POSITIONER <REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION>	1653 lbs	2416 btu
⑪	1	UPS INTERFACE BOX		
⑫	1	ATLAS CABINET <C2>	659 lbs	1825 btu
⑬	1	ATLAS CABINET <C1>	1115 lbs	3389 btu
⑭	1	UPS CABINET	1170 lbs	4061 btu
⑮	1	WATER CHILLER	449 lbs	18716 btu
⑯	1	IVUS VOLCANO S51 CONSOLE, INCLUDES FLAT PANEL MONITOR AND KEYBOARD <DESK MOUNTED>	68 lbs	1631 btu
⑰	1	IVUS VOLCANO COLOR PRINTER		
⑱	1	18 in. MONITOR ON WALL SUPPORT	26 lbs	204 btu
⑲	1	CONTROL ROOM MONITOR WITH DL KEYPAD	22 lbs	204 btu
⑳	1	OPERATORS CONSOLE	22 lbs	546 btu
㉑	1	BOLUS CHASE HANDSWITCH	2 lbs	
㉒	1	AW WORKSTATION	81 lbs	1201 btu
㉓	1	FOUR MONITORS & SURGICAL LAMP SUSPENDED FROM ARM/ CEILING SUPPORT		
㉔	1	TWO MONITORS & TWO SURGICAL LAMPS SUSPENDED FROM ARM/ CEILING SUPPORT		
㉕	1	TWO MONITORS, RADIATION SHIELD & SURGICAL LAMP, SUSPENDED FROM ARM/ CEILING SUPPORT		
⑤①	1	INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'PDB' ON SHEET E1 FOR DETAILED DESCRIPTION.	326 lbs	1532 btu

ENVIRONMENT

Ambient Operation Temperature: 55-75° (F)  
 Allowable Temperature Change: 15° (F) per hour  
 Humidity: 30%-70% non condensing  
 Allowable Humidity Change: 10% per hour  
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE 5-93

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
60		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
61		MED GASES IN CEILING		
62		LEAD APRON RACK		
63		COUNTER TOP WITH BASE AND WALL CABINETS		
64		COUNTER TOP FOR EQUIPMENT- MINIMUM DEPTH 30 in. OR ADDITIONAL SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
65		CONTROL WALL TO CEILING WITH LEAD GLASS WINDOW		
66		CATHETER CABINETS		
67		SHELF - CUSTOMER TO PROVIDE ADEQUATE WALL SUPPORT		
68		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
69		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
70		150-AMP DISCONNECT BREAKER <FOR LOCK-OUT/TAG-OUT CAPABILITY>		
71		CUSTOMER SUPPLIED STORAGE CABINET		
72		RACK FOR HYBRID NETWORK ITEMS		
73		PROCEDURAL CAMERA		
74		PROCEDURAL VIDEO MONITOR		
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
90		X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -CAT. NO. E4502SS FOR WARNING LIGHT & ROOM LIGHT CONTROL.		